



Saliva Sidekick™ Saliva Ejector Tube Holder

1. When sterilized and ready to use, position your saliva ejector tube under the tube holder and press it up into the holder until it is secure.
2. Slide the Saliva Sidekick into the magnetic track on the Bite Buddy, ensuring that it is securely in place.
3. Position the Bite Buddy with the Saliva Sidekick into the mouth according to the Bite Buddy instructions.
4. The Saliva Sidekick can be removed with the Bite Buddy or slid from the track while the Bite Buddy remains in the patient's mouth. Without pulling upwards or downwards, gently slide the saliva ejector tube with the Saliva Sidekick straight out of the Bite Buddy track.

Sterilization Instructions:

Remove from packaging and sterilize all products according to these instructions before use.

Sterilization should be performed using chemical methods or via autoclave. Chemical sterilization is the recommended method of sterilization for added longevity of your products, follow sterilization instructions of your sterilant liquid for chemical sterilization.

Autoclave Sterilization:

Please note: Insufficient water to the autoclave could cause parts to melt.

1. Products should be cleaned as soon as possible after use to prevent soils from hardening. Remove all visible debris prior to sterilizing.
2. Process product through steam autoclave following the autoclave manufacturer's instructions. Do not exceed a temperature of 130°C/266°F.
3. Store only in FDA approved sterilization pouch.

The Saliva Sidekick is latex free.

Be sure to take all necessary precautions to avoid dislodgement or aspiration by following the instructions in this pamphlet.

Ascentcare's products are protected by US Patents. For patent information, visit:
www.ascentcaredental.com/patents

Ascentcare
Dental Products

335 North Griffin Street, Grand Haven, MI 49417
(616) 600-4505
www.AscentcareDental.com