

Ascentcare

Dental Products



Feather Flex™ Kit

SKU: FFK5



Feather flex kit

Instructions for Use:

1. Connect your VacuLUX™ HVE Adapter, VacuVUE™ HVE Mirror or suction tip to the aluminum coupler.
2. Connect the stainless steel tube to your HVE valve.
3. Dock your HVE valve at your docking station.
4. When not in use, rest the vacuum line in the clip to securely hold your instrument.

Sterilization Instructions:

Do not put in autoclave. Run the Feather Flex™ vacuum line with a non-corrosive, pH-neutral enzymatic cleaning solution and perform a surface disinfection with a hospital grade disinfectant wipe between each patient.

Visit our website to learn more about your Feather Flex™ Kit
& compatible dental instruments.

www.AscentcareDental.com

335 North Griffin Street, Grand Haven, MI 49417
support@ascentcaredental.com

(616) 600-4505