



## Bite Buddy™

### Mouth Prop

#### **Instructions:**

1. When sterilized and ready to use, floss may be tied around the Bite Buddy for quick retrieval if needed.
2. Place in mouth over back molars using Placement Pal or index finger.
3. Do not place over front teeth.
4. To remove, use Placement Pal or index finger.



335 North Griffin Street  
Grand Haven, MI 49417

**(616) 600-4505**

[www.AscentcareDental.com](http://www.AscentcareDental.com)

## Sterilization Instructions:

Remove from packaging and sterilize all products according to these instructions before use.

#### **Autoclave Sterilization:**

Please note: Insufficient water to the autoclave could cause parts to melt.

1. Products should be cleaned as soon as possible after use to prevent soils from hardening. Remove all visible debris prior to sterilizing.
2. Process product through steam autoclave following the autoclave manufacturer's instructions. Do not exceed a temperature of 130°C/266°F.
3. Store only in FDA approved sterilization pouch.

The Bite Buddy is latex free.

As with all products containing magnets, please take care to avoid inadvertently deactivating such devices as pacemakers, ICD's and other implanted medical devices.

Be sure to take all necessary precautions to avoid dislodgement or aspiration by following the instructions in this pamphlet.

Ascentcare's products are protected by US Patents. For patent information, visit:  
[www.ascentcaredental.com/patents](http://www.ascentcaredental.com/patents)